
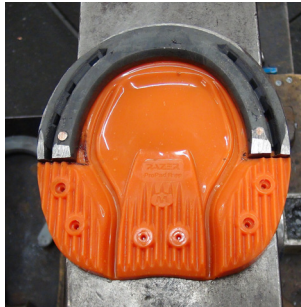


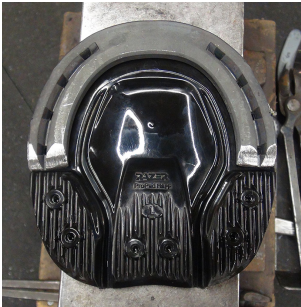

Razer ProPad Flapp Pads Available at an FPD Dealer Near You

The Razer ProPad Flapp pads are made of polyurethane and mimic the natural function of the hoof while still offering protection. Available in sizes Large and Medium.

FEATURES

- Supports the bony column
- Reduces stress to the laminae and wall
- Reduces pressure to the joints and ligaments

BENEFITS

- Increased comfort due to shock absorption
- Proper weight distribution removing weight from the hoof wall
- Increased blood flow due to appropriate frog-to-ground contact



Watch a Video Featuring Razer Flapp Pads on YouTube @farrierproducts

Dave Farley, CF APF-I visited the FPD FootPro™ Shop and shared some good tips on how to get the most effective set up with Razer Flapp pads. The resulting video, "Tips on Using a Razer Flapp Pad" gives useful tips on how to set up your Razer Flapp Pads. View online at https://youtu.be/bttZ9X_KOb0.

Find an FPD dealer near you at farrierproducts.com/locations and ask about Razer products.

JUST A REMINDER



AS SEEN ON FPD'S FACEBOOK PAGE

National Trim Week

Recently, FPD dubbed #NationalTrimWeek on Facebook, sharing video and still shots from the top hands in the farrier trade with a focus on the trim process.

Find photos and video of Bobby Menker, CJF APF-I and Dave Farley, CF APF-I working through the trim process, as well as helpful information on how to tackle the unexpected hoof at www.facebook.com/FarrierProductDistribution.

FPD 20 for 2020 Facebook Giveaway

THE AUGUST 2020 FACEBOOK GIVEAWAY WILL BE ONE CARTRIDGE OF VETEC SOLE-GUARD AND (2) TIPS



Follow FPD on Facebook @farrierproductdistribution to see what we're offering in the current month and look for posts like this one to register for a chance to win the monthly free give-away. This giveaway is only valid in the U.S. and Canada.

Proper Vettec 210 Cartridge Tip Application

There have been a number of questions from individuals who were not aware of the proper mounting of tips on the Vettec 210 cartridges. Below are some images to help clear up any questions you may have. Be sure to like and follow FPD on Facebook @farrierproductdistribution for educational videos on this topic.




1 Cartridge just out of package. Green plug is secured by the locking collar.



2 Twist the locking collar a quarter turn to remove the green plug. It's best to keep the cartridge upright during the tip application process.



3 The tip is indexed so it only fits on the cartridge one way. Don't try to force it if you don't have it in the right position. Place the tip on the cartridge.

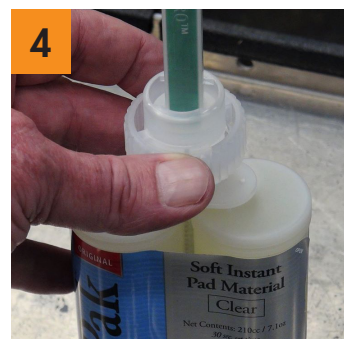

4 Now slide the locking collar over the tip and twist a quarter turn to lock the tip on.



5 After finishing application, remove the tip and put the green plug back on the cartridge. It is also indexed and only fits one way. This keeps the ports from being cross contaminated and causing any problems for your next application.



6 Put the locking collar on- twist a quarter turn and the cartridge can now be stored without any risk of leaking.

[HOME](#)
[ABOUT](#)
[PRODUCTS](#)
[CONTACT](#)
[FAQ](#)

MORE VALUE FOR YOUR MONEY. 30% MORE.

The new 210cc Vettec cartridge features a leak-free design, holds 30% more material and is easier to open and dispense. Bigger. Better. Lower Price.

WHERE TO BUY

Farrier Product Distribution, Inc. (FPD) is now the exclusive distributor of Vettec equine products in the U.S. and Canada through their dealer network. To find a Vettec dealer near you, enter your zip or postal code below.

CHOOSE ONE OPTION

ENTER YOUR ZIP OR POSTAL CODE

FIND MORE TIPS AND
PRODUCT VIDEOS ONLINE
AT [FARRIERPRODUCTS.COM/
VETTEC](http://FARRIERPRODUCTS.COM/VETTEC)